




FRED easy

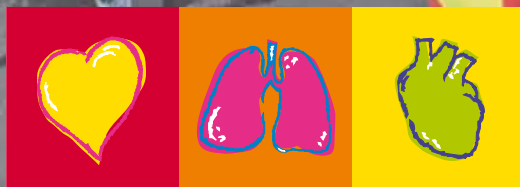

The Automated External Defibrillator
that accompanies all first aid rescuers in
their life-saving mission



SCHILLER

The Art of Diagnostics

FRED easy

The FRED easy Automated External Defibrillator is the best life-saving tool for all emergency services: paramedics, ambulances, firefighters....

It is light, rugged and user friendly and therefore adapted to all situations.

FRED easy features the unique pulsed biphasic waveform MULTIPULSE BIOWAVE with patient impedance compensation. This waveform is efficient for adults and children thanks to the automatic recognition of children pads and energy adaptation.

Adapted voice and displayed instructions for first aid rescuers together with a fast rhythm analysis and shock delivery guide the rescuer during the resuscitation process and minimise the hands-off time.

In order to follow the patient throughout the resuscitation process, the intervention data is recorded and can be reviewed on a PC and a report can be generated.

FRED easy is available in a range of versions and options to fulfil the rescuer's needs:

- Semi or fully automatic
- Metronome: providing the correct rhythm for chest compressions
- ECG display
- Manual override: doctors can switch off the AED mode and decide whether they want to defibrillate or not




FRED easy with manual override



FRED easy fully automatic



ECG display

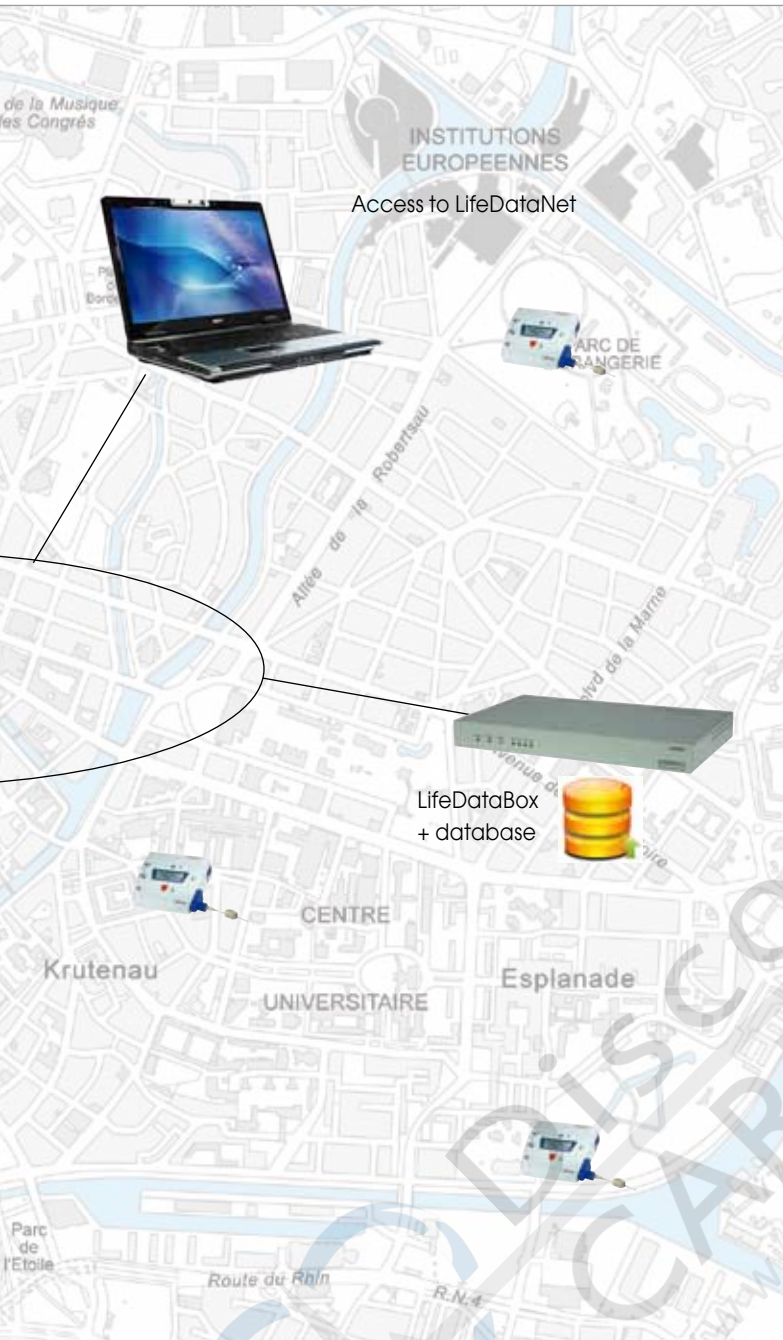

FRED easy Online

The first defibrillator featuring automatic data transmission. Intervention data and the device configuration can be sent to a server via an Ethernet network.

Advantages for everyone:

Transmission is easy, fast and safe. The data is stored centrally for direct and immediate access by authorised persons from any computer connected to the network. Multi-user access to the data via any computer connected to the network (hospital, treating physician, cardiologist, rescue services, administration). This system considerably simplifies the handling of medical and administrative patient data (no paper, no memory card).

LifeDataNet:

The transmitted data is stored on a central server, LifeDataBox, with the LifeDataNet application.

Two types of data can be transmitted:

- Patient data (events, ECG curve, patient impedance)
- Configuration and maintenance information

The LifeDataNet application is a simple and user-friendly interface enabling you to:

- see which device has transmitted the data (serial number, identification, transmission date and time)
- download and analyse patient data
- view the device settings (mode, energy settings, options)
- view the maintenance status (self-test result, battery status, software version).

When a first aid vehicle comes back to the station, simply connect the FRED easy to the network and the data is automatically downloaded and the device ready for a new intervention.

Schiller also offers ARGUS PRO LifeCare 2 and DG4000 to perform advanced patient resuscitation and monitoring.



ARGUS PRO LifeCare 2



DEFIGARD 4000



Asia
SCHILLER Asia-Pacific / Malaysia
 52200 Kuala Lumpur, Malaysia
 Phone +603 6272 3033
 Fax +603 6272 2030
 sales@schiller.com.my
 www.schiller-asia.com



Austria
SCHILLER Handelsgesellschaft m.b.H.
 A-4040 Linz
 Phone +43 732 709 90
 Fax +43 732 757 000
 sales@schiller.at
 www.schiller.at



China
SCHILLER Int'l Trading (Shanghai) Co. Ltd.
 200335 Shanghai, China
 Phone +86-21-62099627
 Fax + 86-21-62099623
 sales@schiller.com.my
 www.schiller.cn



France
SCHILLER Médical S.A.S.
 F-77162 Wissembourg/Cedex
 Phone +33 3 88 63 36 00
 Fax +33 3 88 94 12 82
 info@schiller.fr
 www.schiller-medical.com



France (distribution France)
SCHILLER France S.A.S.
 F-77600 Bussy St Georges
 Phone +33 1 64 66 50 00
 Fax +33 1 64 66 50 10
 infoschiller@schiller-france.fr
 www.schiller-france.com



Germany
SCHILLER Medizintechnik GmbH
 D-85521 Ottobrunn
 Phone +49 89 62 99 81-0
 Fax +49 89 609 50 90
 info@schillermed.de
 www.schillermed.de



Hungary
SCHILLER Diamed Ltd.
 H-1141 Budapest
 Phone +36 (1) 383-4780 / 460-9491
 Fax +36 (1) 383-4778
 sales@schiller.at
 www.schiller-hungary.hu



India
SCHILLER Healthcare India Pvt. Ltd.
 Mumbai - 400 001, India
 Phone +91 22 6152 3333/ 2920 9141
 Fax +91 22 2920 9142
 sales@schillerindia.com
 www.schillerindia.com



Japan
SCHILLER Japan, Ltd.
 Hiroshima 734-8551
 Phone +81 82 250 2055
 Fax +81 82 253 1713
 koji.maekawa@schiller.jp
 www.schiller.jp



Poland
SCHILLER Poland Sp. z o.o.
 PL-02-729 Warszawa
 Phone +48 22 8432089
 Fax +48 22 8432089
 schiller@schiller.pl
 www.schiller.pl



Russia & C.I.S.
SCHILLER AG Rep. office
 125124 Moscow, Russia
 Phone +7 (495) 970 11 33
 Fax +7 (495) 970 11 33
 mail@schiller-ag.com
 www.schiller-cis.com



Spain
SCHILLER ESPAÑA, S.A.
 E-28230-Las Rozas/Madrid
 Phone +34 91 713 01 76
 Fax +34 91 355 79 33
 schiller@schiller.es
 www.schiller.es



Switzerland
SCHILLER-Reomed AG
 CH-8953 Diefikon
 Phone +41 44 744 30 00
 Fax +41 44 740 37 10
 sales@schiller-reomed.ch
 www.schiller-reomed.ch




Turkey
SCHILLER TÜRKIYE
 Okmeydanı-Sisli - İstanbul
 Phone +90 212 210 8681 (pbx)
 Fax +90 212 210 8684
 sales@schiller-turkiye.com
 www.schiller-turkiye.com



USA
SCHILLER America Inc.
 Doral, Florida 33172
 Phone +1 786 845 0620
 Fax +1 786 845 06 02
 sales@schilleramerica.com
 www.schilleramerica.com

11/11

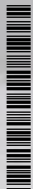

Part no.: 2.500378



SCHILLER

The Art of Diagnostics