

M11/M9/M3

Automatic Sterilizers



**Ritter**<sup>®</sup>  
by MIDMARK

# M11 & M9 Sterilizers


## Ritter M11 and M9 UltraClave® Automatic Sterilizers

Today more than ever, you need products that are reliable and simple to use. Ritter sterilizers couldn't be easier and are time tested for reliability. With any Ritter sterilizer, simply load it, set it and forget it. Select the cycle and the time is set for you. Press "Start" and the sterilizer will fill with the correct amount of water. Once the sterilization process is complete, the door opens automatically and quietly to dissipate the steam, giving you a fast drying time for your instruments. Less complexity means better reliability and a cost effective solution. It's real convenience.



Both the Ritter M11 and M9 sterilizers provide easy access front fill and drain, and easy-to-understand controls.

The SteamFlush Pressure Pulse Air Removal System is so efficient that it can pass the air removal tests required on more complex vacuum steam sterilizer systems, ensuring good steam penetration for proper sterilization. The M11 is the largest of the Ritter automatic sterilizers – its unique 11" x 18" chamber gives it the distinction of the largest sterilizing capacity of any standard countertop sterilizer on the market, making it perfect for larger packs and instruments. An optional printer accessory, available on both the M11 and M9, provides records of the cycle, cycle time, temperature and pressure.

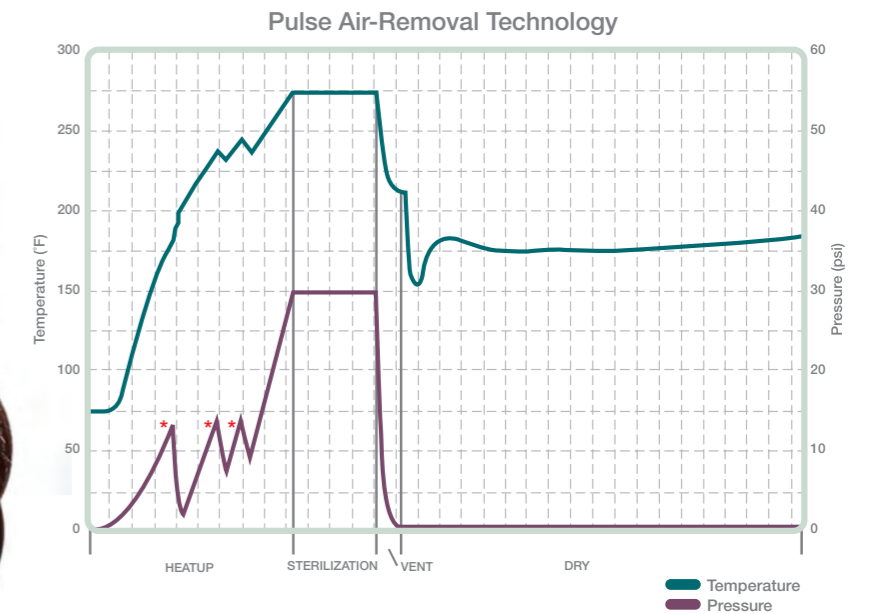

\*Three air removal purges.  
New electronic bellows purge air from chamber three times prior to sterilization for efficient air removal.

Symbol	Cycle Type	Hot Cycle Times*		Cold Cycle Times*		Sterilization Time		Sterilization Temp. & Pressure	
		M9	M11	M9	M11	M9	M11	M9	M11
	Unwrapped	14 min.	18 min.	19 min.	28 min.	3 min.	3 min.	132° C 270° F 27.1 psi (186 kPa)	132° C 270° F 27.1 psi (186 kPa)
	Pouches	17 min.	22 min.	22 min.	32 min.	5 min.	5 min.	132° C 270° F 27.1 psi (186 kPa)	132° C 270° F 27.1 psi (186 kPa)
	Packs	40 min.	44 min.	46 min.	56 min.	30 min.	30 min.	121° C 250° F 15 psi (104 kPa)	121° C 250° F 15 psi (104 kPa)

\*Cycle times do not include drying time.  
Cycle times are approximate and may vary depending on instrument loads.  
Preset drying time is 30 min. but can be programmed from 0 to 60 minutes.



**Cycle Start Button**  
(Initiates selected cycle)

**Manual Stop Button**  
(Terminates selected cycle or function)

**Self Program Controls**

Programmable controls allow for creation of different cycle parameters to meet your unique sterilization needs. Change the following: Time (3 to 90 minutes), Temperature (230 to 275°F), Dry Time (0 to 60 minutes), Vent (slow or fast) and two program buttons. The #1 and #2 buttons allow you to store the change if desired.

**Pre-Programmed Controls**

Four pre-programmed sterilization cycle controls: Unwrapped (132° C or 270° F for 3 minutes), Pouched (132° C or 270° F for 5 minutes), Packs (121° C or 250° F for 30 minutes), Handpiece (For sterilization of lumened devices and/or dental handpieces: 132° C or 270° F for 6 minutes).

**LCD Display**

Indicates cycle selected, temperature and exposure time for selected cycle. During the cycle, the display shows messages describing status of cycle. When the cycle enters sterilization mode, remaining time is displayed as well as temperature and pressure.

**The Right Thing to Do For You, Your Staff and Your Practice**

You and your staff's time is precious, so why waste it? Break free from your sterilizer with the reliable, efficient Ritter M11 and M9 UltraClave® Automatic Sterilizers and spend your time on more important tasks. Make the industry-leading sterilizers your choice for sterilization and have peace of mind.



The Midmark Steri-Center offers the optimum in sterilization efficiency and storage. With its single direction instrument flow, there are no crossover storage issues to contaminate sterile instruments (available in 8', 10' or 12' configurations). For all your casework needs, please ask for our full-line Midmark casework brochure.

**M3 Standard Features**



**Door Tray**  
Simple, one-step tray operation. Lightweight, durable stainless steel. Ergonomically designed handle for balance and comfort.



**Top Fill**  
Convenient access to water reservoir. Water level sensor provides intelligent system monitoring and feedback.



**Drain Access Panel**  
Front reservoir drain with on-board drain tube storage is practical and time-saving.



**Quick Reference Guide and CD**  
On-board user instructions and interactive training CD provide handy, instant information.



**Condensate Tank**  
Traps and contains expelled steam. Quick release lid, convenient carrying handle and water level sensor for efficient, worry-free use.

# M3 Sterilizer



**Ritter M3 UltraFast® Automatic Sterilizer**

The Ritter M3 UltraFast® Automatic Sterilizer is everything you need in an autoclave. Sterilize unwrapped instruments in 6 minutes, pouched instruments in just over 10, with the touch of a button. The M3 is easy to use with a one-step loading operation, push-button cycle selection and automatic door. Most importantly, your instruments are dry and ready to use at the end of every complete cycle.

The Ritter family of sterilizers is an all-inclusive solution for your sterilization needs. And, just like the rest of the family, the M3 comes standard with the dependability and reliability that you have come to expect from the Ritter name.

Symbol	Pre-programmed Cycle	Sterilization Temperature & Pressure	Hot Cycle Time (Heat-up & Vent)	Sterilization Time	Total Hot Cycle Time	Dry Time	Total Hot Cycle Time w/ Dry
	Unwrapped	270° F 132° C 27.1 psi (186 kPa)	2.5 min.	3.5 min.	6 min.	25 min.	31 min.
	Pouches	270° F 132° C 27.1 psi (186 kPa)	5 min.	5.5 min.	10.5 min.	30 min.	40.5 min.
	Low Temp	250° F 121° C 15 psi (104 kPa)	4.5 min.	20 min.	24.5 min.	50 min.	74.5 min.

## Accessories that add convenience.

### M11 & M9 Accessories



#### **Impact Printer**

Records and prints critical sterilization cycle data, including chamber temperature and pressure, cycle elapsed times, and program cycle type.



#### **Pouch Rack**

Conveniently separates sterilization pouches for enhanced steam circulation and drying.



#### **Deep Tray**

Conveniently houses larger instruments for sterilization.



#### **External Condensing Tank**


When single use distilled water is desired for all sterilization cycles, the External Condensing Tank is a must.

### M3 Accessories



#### **Top Cover Protector**

Use the cover protector to shield the M3 top from scratches or damage.

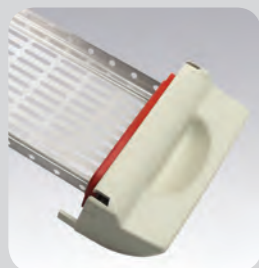

#### **Door Tray**

Additional door trays allow higher efficiency instrument processing.



#### **Printer**

The standalone printer is optimal for mechanical monitoring of each cycle. Records and prints critical sterilization cycle data, including chamber temperature and pressure, cycle elapsed times, and program cycle type.

## What do you want in a sterilizer?

- Fast and easy use for efficient instrument processing
- Reliable and effective for quality infection control
- Maximum space utilization and a complementary design for your sterilization area

Our full line of sterilizers address all of your critical cleaning needs. In fact, we're sure you'll find a Ritter sterilizer that's just right for your practice.



## Ritter M11, M9, M3

### S P E C I F I C A T I O N S

#### Ritter M11 Specifications

Overall length w/plug: 23.8" (60.5 cm)  
Overall width: 17.8" (45.2 cm)  
Overall height w/printer: 17.8" (45.2 cm)  
Minimum Countertop area:  
17.8" x 21.0" (45.2 cm x 53.3 cm)

#### Chamber:

11" diameter (28 cm)  
18" depth (45.7 cm)  
6.5 gal usable volume (24.6 liters)

#### Trays:

Two large – 9" W x 15" L x 1.1" D  
(22.9 cm x 38 cm x 2.9 cm)

Two small – 6.6" W x 15" L x 1.1" D  
(16.8 cm x 38 cm x 2.9 cm)

Unit weight: 99 lb (44.9 kg)

Shipping weight: 131 lb (59.4 kg)

Water reservoir capacity:  
1.4 gal (5.3 liters)

#### Electrical Requirements

115 VAC, 50/60 Hz, 15 amps  
Single Phase

115 VAC has 1425 watt tubular  
immersion heater

UL61010-1, IEC 61010-2-040

CAN/CSA C22.2 No. 61010-1

CSA C22.2 No. 61010-2-040-07  
Part 2-040

Meets the requirements of ASME Boiler  
and Pressure Vessel Code

Canadian Registration Number (CRN)  
available

A separate (dedicated) circuit is  
recommended for this sterilizer

#### Optional Accessories

9A259001 - Impact Dot Matrix Printer  
9A225001 - Sterilizer Deep Tray  
9A226001 - Sterilizer Pouch Rack  
9A307001 - Cool Hand Tool  
(Tray & Cassette Removal Tool)  
9A260001 - External Condensing Tank  
9A215001 - Horizontal Cassette Rack  
9A215002 - Vertical Cassette Rack

#### Ritter M9 Specifications

Overall length w/plug: 20.4" (51.8 cm)  
Overall width: 15.3" (38.9 cm)  
Overall height w/printer: 15.8" (40.1 cm)  
Minimum Countertop area:  
15.3" x 17.9" (38.9 cm x 45.4 cm)

#### Chamber:

9" diameter (22.9 cm)  
15" depth (38.1 cm)  
3.5 gal usable volume (13.4 liters)

#### Trays:

Two large – 7.3" W x 12" L x .8" D  
(18.6 cm x 30.5 cm x 2.2 cm)

Two small – 5.6" W x 12" L x .8" D  
(14.3 cm x 30.5 cm x 2.2 cm)

Unit weight: 73 lb (33.1 kg)

Shipping weight: 81 lb (36.7 kg)

Water reservoir capacity:  
1.1 gal (4.1 liters)

#### Electrical Requirements

115 VAC, 50/60 Hz, 15 amps  
Single Phase

115 VAC has 1425 watt tubular  
immersion heater

UL61010-1, IEC 61010-2-040

CAN/CSA C22.2 No. 61010-1

CSA C22.2 No. 61010-2-040-07  
Part 2-040

Meets the requirements of ASME Boiler  
and Pressure Vessel Code

Canadian Registration Number (CRN)  
available

A separate (dedicated) circuit is  
recommended for this sterilizer

#### Optional Accessories

9A259001 - Impact Dot Matrix Printer  
9A224001 - Sterilizer Deep Tray  
9A226001 - Sterilizer Pouch Rack  
9A307001 - Cool Hand Tool  
(Tray & Cassette Removal Tool)  
9A260001 - External Condensing Tank

#### Ritter M3 Specifications

Overall length w/plug: 22" (55.9 cm)  
Overall width: 17.8" (45.2 cm)  
Overall height: 6.9" (17.5 cm)  
Minimum Countertop area:  
24" D x 22" W (61 cm x 55.9 cm)

#### Chamber:

12.1" L x 7.6" W x 1.6" H (30.7 x 19.3 x 4.1 cm)  
0.49 gal usable volume (1.8 liters)

#### Tray:

7.25" W x 11.75" L x 1" D (29.8 x 18.4 x 2.5 cm)  
Holds the Hu-Friedly® Signa-Stat  
Cassette (6.5" x 10.5" x 1.25")

Unit weight: 71 lb (32.2 kg)

Unit weight with water: 80 lb (36.3 kg)

Shipping weight: 80 lb (36.3 kg)

Water reservoir capacity:  
1.2 gal (4.5 liters)

#### Electrical Requirements

115 VAC, 50/60 Hz, 12 amps  
Single Phase

230 VAC, 50/60 Hz, 6 amps  
Single Phase

Boiler heater power: 1400 watts

Maximum Power Consumption: 1400 watts

UL 61010-1, 2nd Edition, IEC 61010-2-040

CAN/CSA C22.2 No. 61010-1 2nd Edition

Meets the requirements of ASME Boiler  
and Pressure Vessel Code

Canadian Registration Number (CRN) available

A separate (dedicated) circuit is recommended  
for this sterilizer

#### Optional Accessories

9A404001 - Top Cover Protector  
9A402001 - Door Tray  
9A401001 - Printer



M550, M250 and M150 Soniclean® Ultrasonic Cleaners

Midmark is an ISO 13485 and  
ISO 9001 Certified Company.

For more information  
or a demonstration, contact  
your Ritter dealer or  
Midmark Corporation,  
Versailles, Ohio 45380.  
Call: 1-800-MIDMARK  
Fax: 1-800-365-8631  
midmark.com

**Ritter**®  
by MIDMARK