


# Ambu® Aura40 Straight Reusable Laryngeal Mask

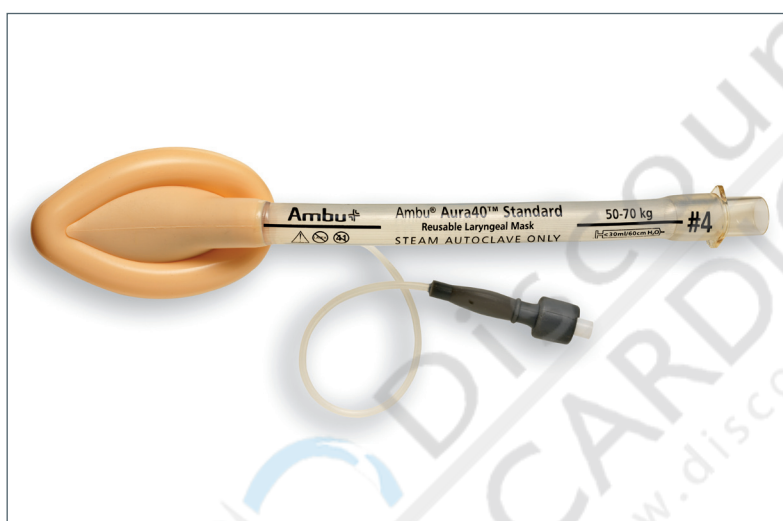

The Aura40 Straight is intended for use as an alternative to the face mask for achieving and maintaining control of the airway during routine and emergency anesthetic procedures in fasted patients.

Aura40 Straight is a reusable laryngeal mask that is autoclavable up to 40 times. The absence of epiglottis bars ensures easier tracheal access.

The mask cuff is designed to conform to the contours of the hypopharynx. The Aura40 Straight is available in 8 sizes, ranging from neonate through adult.

## KEY FEATURES


- Reusable
- Available in 8 sizes from neonate to adult
- Clearly marked guidelines for easy insertion, cuff inflation volume, and patient weight listed on the airway tube
- Autoclavable up to 40 times
- Latex Free

## AURA40 STRAIGHT MATERIAL

AURA40 STRAIGHT	MATERIAL
Tube/Cuff	Silicone
Airway Connector	Polysulphone
Pilot Tube	Silicone
Pilot balloon with Inflation Valve	Silicone/PP/SS

## ENVIRONMENT AURA40 STRAIGHT AND PACKAGING

PVC - Free	Yes
DEHP - Free	Yes
Latex-Free	Yes

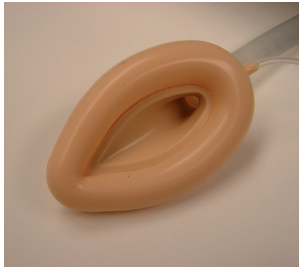

## SPECIFICATIONS

	Infant		Pediatric		Adult			
	#1	#1½	#2	#2½	#3	#4	#5	#6
Mask size	#1	#1½	#2	#2½	#3	#4	#5	#6
Airway connector	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)	15 mm male (ISO)
Min. I.D. Tube	5.2 mm	6.6 mm	7.0 mm	8.5 mm	9.8 mm	9.8 mm	11.5 mm	11.5 mm
Max. O.D. Tube	8.0 mm	9.8 mm	10.6 mm	12.7 mm	14.8 mm	14.8 mm	16.5 mm	16.5 mm
Inflation valve	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)	Luer cone (DIN 13090 F6)

Appropriate Storage Temperature -40°C to 60°C

## SELECTION GUIDELINES

	Infant		Pediatric		Adult			
	#1	#1½	#2	#2½	#3	#4	#5	#6
Mask size	#1	#1½	#2	#2½	#3	#4	#5	#6
Patient weight	< 5 kg	5-10 kg	10-20 kg	20-30 kg	30-50 kg	50-70 kg	70-100 kg	>100 kg
Maximum cuff inflation volume	4 ml	7 ml	10 ml	14 ml	20 ml	30 ml	40 ml	50 ml
Maximum intra-cuff pressure	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O	60 cm H <sub>2</sub> O

## ORDERING SPECIFICATIONS

Item number	Size	Type
340 100 000	1	Reusable Straight Laryngeal Mask
340 150 000	1½	Reusable Straight Laryngeal Mask
340 200 000	2	Reusable Straight Laryngeal Mask
340 250 000	2½	Reusable Straight Laryngeal Mask
340 300 000	3	Reusable Straight Laryngeal Mask
340 400 000	4	Reusable Straight Laryngeal Mask
340 500 000	5	Reusable Straight Laryngeal Mask
340 600 000	6	Reusable Straight Laryngeal Mask



**Ambu** 

Ambu Inc.  
6740 Baymeadow Drive  
Glen Burnie, MD 21060  
Tel.: 1 410 768 6464  
1 800 262 8462  
Fax: 1 800 262 8673  
www.ambu.com

493 3200 31 - 08/2007 - Ambu AS - Technical data may be modified without further notice.